**Assista-Rail™**

AST-S  
(DC111 - RevE)

[healthcraftproducts.com](http://healthcraftproducts.com)

## **WARNING FOR BED SAFETY**

### **WARNING**

#### **ENTRAPMENT HAZARD**

There is a risk of entrapment associated with all bedside mobility aids. Entrapment occurs when an individual becomes wedged between a bed rail and another object (example: mattress). Entrapment can result in serious injury and death. Read and understand the enclosed Entrapment Prevention Guidelines document. See the additional Entrapment Prevention Guidelines:

- [www.fda.gov/bedsafety](http://www.fda.gov/bedsafety)
- <https://www.cpsc.gov/safety-education/safety-guides/furniture-furnishings-and-decorations/adult-bed-rails>


**PRIOR TO INSTALLATION**, review all of the product specifications and instructions to ensure that it is safe to use with your bed and mattress.

(a) This product is equipped with a permanently attached safety strap to securely install the product to a metal bed frame. **NEVER** remove this safety strap.

(b) This product is intended to be installed to a metal bed frame. Never use this product without a bedframe. Never use this product on adjustable beds.

(c) **PRIOR TO EVERY USE**, ensure there is no gap between the product and the side of the mattress. If there is a gap, the product was improperly installed. **Stop using immediately if product shifts from its original position, if product is damaged, broken or missing parts.**

(d) Increase risk of entrapment when using bed rails with a soft mattress. Test mattress by applying 12lbs (5.4kg) of weight on the edge of the mattress. If the mattress can compress by 2" (51cm), do not use this product.

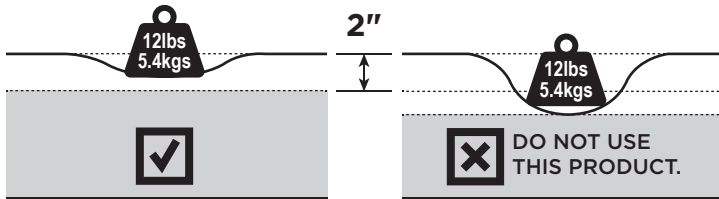(e) If product is used in a care facility, follow the installations policies of the institutions. If this product does not comply with those policies, do not install this product.

(f) When changing bed linens, always confirm that product is installed correctly as per installation instructions. Re-tighten the safety strap to eliminate a gap between product and side of mattress.

For questions about product safety or installation, or in case of a serious event with this product, contact HealthCraft at 1-888-619-9992 immediately. In addition to contacting the manufacturer directly, consumers should report problems to the CPSC at this website: <https://www.saferproducts.gov/> or call 1-800-638-2772, or to the FDA at 1-800-332-1088.

### **WARNING**

#### **ENTRAPMENT HAZARD**

**NEVER** use product without properly securing it to bed. Incorrect installation can allow product to move away from mattress, bed frame and/or head or foot boards, which can lead to entrapment and death.

### **WARNING**

#### **SUFFOCATION, STRANGULATION, ENTRAPMENT HAZARD**

If product is installed incorrectly or moves from its initial position gaps can occur which can entrap and cause fatality. People with Alzheimer's disease, dementia or other neurological conditions, or those who are sedated, confused, or frail, are at increased risk of entrapment, suffocation and strangulation.

- **NEVER** use unless product is tight against mattress, without gaps, and at least 12-1/2 in. from headboard and footboard.
  - **NEVER** use with children.
  - **NEVER** use on toddler, bunk, water, electrical or inflatable beds, or on beds with mattress toppers or soft compressible pads.
  - **DO NOT USE** if bedside organizer is removed from rail.
- This product is under the jurisdiction of the FDA; to report problems or injuries, call 1-800-332-1088.

### **WARNING**

#### **ENTRAPMENT, STRANGULATION, SUFFOCATION AND FALL HAZARDS**

Gaps in and around this product can entrap and kill. People with Alzheimer's disease or dementia, or those who are sedated, confused, or frail, and are at increased risk of entrapment and strangulation. People attempting to climb over this product are at increased risk of injury or death from falls. Always make sure this product is properly secured to bed. If product can move away from bed or mattress, it can lead to entrapment and death.

### **WARNING**

#### **ENTRAPMENT, STRANGULATION, SUFFOCATION AND FALL HAZARDS**

If the strap provided is not properly secured the product may move into an unsafe position which increases the danger of entrapment. See instructions for proper use of the straps.

# English

## OVERVIEW

Thank you for choosing the Assista-Rail from HealthCraft. Please read and understand the instructions in this manual; keep manual for future reference. It is your responsibility to see that your Assista-Rail is properly assembled, installed, and cared for. Failure to follow instructions in this manual could result in serious injury or death. If you are not equipped to undertake the outlined work, we would recommend that you have your Assista-Rail installed by a qualified contractor. The Assista-Rail is a fixed bed rail. The Assista-Rail is intended ONLY for minimal vertical load bearing to provide bed positioning and seating support for people with reduced mobility. The Assista-Rail is intended for home use. The Assista-Rail is installed between the mattress and box spring, and is secured in place with a safety strap. The Assista-Rail is height adjustable to accommodate the thickness of the mattress. The maximum user weight for this product is 300lbs / 136kgs. Product is not intended to support full body weight. This product is not intended as a physical constraint or barrier to exiting the bed. The Assista-Rail is not to be used in any other way than described above.

## LIMITED LIFETIME WARRANTY

Products are covered by a limited lifetime warranty against defects in materials and workmanship for the original purchaser. Warranty excludes products that have been damaged through misuse, accidental damage, alteration, normal wear and tear, wood material and stain, or the use of corrosive or abrasive cleaning products. Buyer hereby indemnifies, agrees to hold harmless and defend HealthCraft Products Inc. from and against any and all liabilities, claims, (founded and unfounded), losses, damages, costs and expenses (including without limitation consequential damages and reasonable professional fees) resulting from buyers specification, application, or improper use of goods described hereon; buyers omission or neglect. HealthCraft Products Inc. does not assume any liability for damage resulting from services performed by others or faulty installation, misuse or misapplication of goods sold by HealthCraft Products Inc. HealthCraft Products Inc. shall not be liable for prospective profits or special, indirect, or consequential damages, or for the cost of any corrective work done without HealthCraft Products Inc. prior approval. HealthCraft Products Inc. total liability hereunder shall in no event exceed the purchase price of the goods specified hereon. Specifications subject to change without notice.  
**Manufacturer:** HealthCraft Products Inc. 2790 Fenton Road, Ottawa, Canada, K1T 3T7. © 2024 HealthCraft Products Inc.

## DELIVERY CONTENTS / PARTS

### FIGURE A.

- |                               |                      |
|-------------------------------|----------------------|
| 1. Handle + Bag Assembly (x1) | 4. Back Tube (x1)    |
| 2. Upright Tube (x2)          | 5. Safety Strap (x1) |
| 3. Side Tube (x2)             | 6. Locking Pin (x4)  |

## SAFETY CONSIDERATIONS

### FIGURE B.

The Assista-Rail is intended to be used when the bed casters are locked.  
The Assista-Rail is intended for moderate vertical load bearing ONLY.  
The Assista-Rail is not intended for sideward (lateral) support.  
The Assista-Rail is not intended to support the full weight of an individual.  
The Assista-Rail is not intended for children to play or hang onto.  
The Assista-Rail is not to be used on electric or hospital style beds.  
The Assista-Rail is intended to be height adjustable to the mattress thickness. Select the appropriate position (P) for the mattress thickness.  
The Assista-Rail is intended to be placed as per following:  
(A) Side of bed rail must be at least 12.5" / 320mm from end of bed (B) Top of bed rail must be at least 4" / 102.5mm above top of mattress (C) Base of crossbar must be at least 3" / 76mm below top of mattress.

## ASSEMBLY

### FIGURE C.

1. Slide side tubes over back tube and engage spring locks.
2. Slide upright tubes into sides tubes and engage spring locks.
3. Slide handle assembly over upright tubes.
4. Set desired height, align holes and install locking pin.
5. P3 ONLY - cover exposed holes with additional locking pins (x2).

## INSTALLATION

### FIGURE D.

1. While laying in bed, note position of waistline.
2. Slide rail between box spring and mattress, in line with waistline.
3. Loop the strap between the rail and the angle iron of the bed frame.
4. Tighten strap to pull rail tight against the side of the bed. Ensure no gap between rail and mattress.

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